

## EnviroSorb<sup>®</sup> EnviroPad<sup>®</sup> Reusable, Odor Neutralizing Underpad Cleaning Instructions

*\*Follow manufacturers suggested laundering procedures for the washer and dryer*

- Do not exceed 85% of the washer's load capacity.
- Use properly diluted **bleach** or **hydrogen peroxide**-based detergents following manufacturer instructions.
- DO NOT use fabric softeners (this may affect the absorbency of the EnviroSorb<sup>®</sup> EnviroPad<sup>®</sup>).
- Wash on normal setting with warm water following the washing machine manufacturer instructions.
- Dry on normal setting following the dryer manufacturer instructions.
- Remove from dryer and allow to cool for 5 minutes before use.
- Do not leave underpads in dryer after cycle is complete.